



# COMPTENS

- ▶ Dual channel (4-lead) analog TENS
- ▶ Three treatment modes (Normal, Burst, Modulated Rate)
  - ▶ Treatment timer (15, 30, Continuous)
  - ▶ Hinged protective cap for intensity dials
  - ▶ Removable plastic belt clip
  - ▶ Locking battery cover
  - ▶ 1-year warranty
- ▶ Comes with 1 pkg of 4 reusable self-adhering electrodes, 1 set 43 p in lead wires, 1 each 9V heavy duty battery, carrying case, and instruction manual
- ▶ Leadwire reorder #LW46SPLE



## Technical Specifications

**Channel** Dual, isolated between channels

**Amplitude** Adjustable, 0 - 80 mA peak into 500 ohm load each channel

**Output Voltage** Adjustable, 0 - 40 V peak into 500 ohm load each channel

**Waveform** Asymmetrical biphasic square pulse

**Pulse Rate** Adjustable, 2 - 150 Hz

**Pulse Width** Adjustable, 30 - 260  $\mu$ S

**Modes** Three (Normal, Burst, Modulation)

**Burst (B)** Bursts occur twice per second. Pulse width adjustable, frequency fixed at 100 Hz

**Modulation (M)** Pulse rate is automatically varied in a cyclic pattern over an interval of nominally 10 seconds (max. 150 Hz) Pulse rate decreases linearly over a period of 4 seconds from the control setting value to a value which is 40% less. The lower pulse rate will continue for 1 second, then increase linearly over a 4 second period to its original value. The original pulse rate will continue for 1 second. The cycle is then repeated.

**Timer** 15 minutes, 30 minutes, or continuous

**Power Supply** One 9-volt battery

**Size** 9.5 x 6.5 x 2.35 cm

**Weight** 115 grams with battery

Distributed by:



217 Industrial Court  
Wabasha, MN 55981  
1-800-363-STIM | www.medi-stim.com  
Fax: 1-651-565-2410

