

**NEW!**

# Dr. MED<sup>®</sup>

## Lumbar Orthosis (LO) Back Support

SUPPLEMENTAL ELASTIC WAIST SUPPORT #DR-B021 INCLUDED.  
(see reverse side for details)


**The Dr. Med Lumbar Orthosis (LO) B026 Back Support immobilizes the spine along the lumbar vertebrae for effective treatment of these medical conditions:**

- Low Back Pain
- Lumbar Strains and Sprains
- Spondylolisthesis
- Arthritis of the Lumbar Vertebra
- Spinal Stenosis
- Post-Surgical Stabilization
- Lumbar Muscle Weakness
- Spondylosis
- Herniated Disc/Disc Disease

# SUPPORT MEETS COMFORT

## Dr. Med B026 Features:

- Rigid anterior and posterior panels provide abdominal lift and lumbar support
- Light-weight breathable composition allows maximum air-flow for all day comfort
- Produces intracavitary pressure to reduce load on the intervertebral discs
- Restricts excessive spinal motion, allowing for soft tissue healing, and helps reduce postoperative pain
- Easy to use double-strap pull system provides customizable compression
- Low profile and close-fit design contours to the body with no gaps, and can be worn under clothing
- Adjustable Velcro front for easy on/off
- Available in 9 or 12 heights to accommodate waists 26 - 52

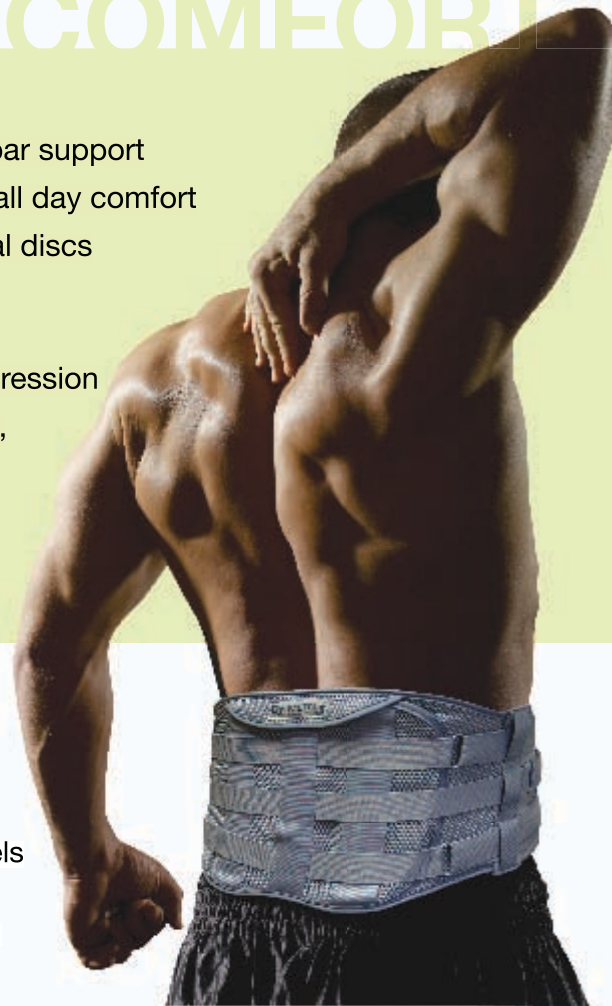

## Low Profile and Comfortable

The Dr. Med B026 offers superior compression and support of the lumbar vertebrae. The double-strap pull system provides adjustable compression that is fully customizable to the patient's needs.



- Pulling the first set of straps fits the panels snugly against the torso.
- Adjusting the center strap adds localized compression to the desired area.

## Removable anterior and posterior panels


As the patient begins to heal, they may not need the panels. Once the panels are removed, the built-in steel stays provide a lighter support for the interim period of the healing process. A hot or cold gel pack can be inserted into the back panel pocket for additional heat or ice therapy.




Suggested Billing Code:  
**HCPC #L0627**

## Easy To Use


B026 easily applied with three simple steps: Place, Fit, and Compress

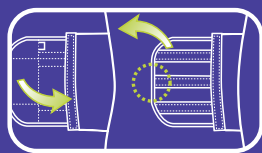


**1**  
Attach front closure over abdomen.

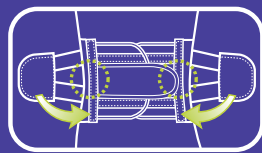

**2**  
Pull double-strap tensioners, to secure anterior panel, and attach to the front.

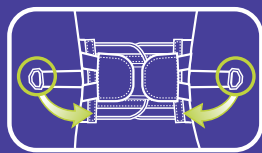

**3**  
Adjust the compression strap to the desired level of support and attach to the front.

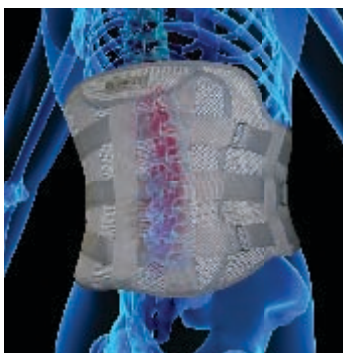

B026 Fiber Content: Polyester 60%, Nylon 35%, Polyurethane 5%

### Care Instructions:

Hand wash with warm water, using mild detergent only, do not bleach. Hang dry. Avoid direct sunlight.

### Warnings:

If a rash or severe itch occurs, please discontinue use and consult your physician. Use caution when pulling the straps to avoid over compression. Do not use the belt for anything other than your physician's prescribed purpose. Over-use of this product may have adverse effects. Avoid use while sleeping. Consult your physician for recommended use.

### Suggested Billing Code:

HCPSC L0267

**Medicare Description:** LUMBAR ORTHOSIS, SAGITTAL CONTROL, WITH RIGID ANTERIOR AND POSTERIOR PANELS, POSTERIOR EXTENDS FROM L-1 TO BELOW L-5 VERTEBRA, PRODUCES INTRACAVITARY PRESSURE TO REDUCE LOAD ON THE INTERVERTEBRAL DISCS, INCLUDES STRAPS, CLOSURES, MAY INCLUDE PADDING, SHOULDER STRAPS, PENDULOUS ABDOMEN DESIGN, PREFABRICATED, INCLUDES FITTING AND ADJUSTMENT

**To qualify the B026 LO Back Support for Medicare billing, one of the following indications must be met:**

- To reduce pain by restricting mobility of the trunk; or
- To facilitate healing following an injury to the spine or related soft tissues; or
- To facilitate healing following a surgical procedure on the spine or related soft tissue; or
- To otherwise support weak spinal muscles and/or a deformed spine.

### INCLUDED BONUS!

### Supplemental Elastic Waist Support # B021

The B021 is included with the B026 to provide a milder form of stabilization during the interim periods when the user cannot wear the B026. This allows the user to maintain some form of stabilization while allowing the back muscles to work so they may maintain their strength and prevent atrophy. The B021 provides mild stabilization and support while allowing the user to go about their normal daily routine, encouraging increased user activity. The B021 is made out of lightweight breathable materials and even the elastic is stitched together in a manner which allows maximum airflow for cool comfort throughout the entire day.



### B021 Features:

- Composed of three breathable materials: cotton, nylon, polyurethane
- Easy-adjust Velcro front with 4 posterior plastic stays for extra support
- Adjustable double-strap elastic tensioners
- 8.25 height



### B026 (9 height sizes)

B0269XS	XS	26 -28	waist
B0269S	S	28 -31	waist
B0269M	M	31 -34	waist
B0269L	L	34 -37	waist
B0269XL	XL	37 -40	waist
B02692X	2XL	40 -44	waist
B02693X	3XL	44 -48	waist
B02694X	4XL	48 -52	waist

### B026 (12 height sizes)

B02612L	L	34 -37	waist
B02612XL	XL	37 -40	waist
B026122X	2XL	40 -44	waist
B026123X	3XL	44 -48	waist
B026124X	4XL	48 -52	waist

### Measuring for Proper Fit:

For waist size, measure around the abdomen over the belly button.  
For height size, measure clavicle to belly button. Use 9 for <14, 12 for >14.



Made From  
Breathable Materials  
Support Meets  
Comfort!

Distributed by:



217 Industrial Court  
Wabasha, MN 55981  
1-800-363-STIM | www.medi-stim.com