

Smart TENS

State-of-the-art

Non-invasive



Safe-start feature



Comfortable

- ▶ Dual channel
- ▶ 9 pre-set treatment regimens
- ▶ MultiTENS mode featuring 8 sequential therapy sessions
- ▶ Fully digital
- ▶ LCD display
- ▶ Treatment compliance timer
- ▶ Control-locking mechanism
- ▶ Sub-pulse and extended pulse technology
- ▶ Stimulation pause & continue button
- ▶ Adjustable intensity button
- ▶ 3-year warranty

Technical Specifications

Channel	Dual		
Amplitude	Max 10.8 mA		
Output Voltage	Max 5.4V		
Waveform	Asymmetrical balanced biphasic		
Pulse Rate	Min 2Hz. - Max 125 Hz. (Depending on program)		
Pulse Width	Min 80 μ S - Max 420 μ S (Depending on program)		
Modes	9 different modes, with additional 8 MultiTENS modes		
	Program Number	Frequency (Hz)	Pulse Width (μ S) Burst Mode
	0 MultiTENS mode	5-125	80-420 none
	1	85	200 none
	2	8	200 Frequency 80 Hz, pulse width 120 μ S for 3 sec. Return to program settings for 1.5 sec.
	3	35	200 none
	4	100	200 none
	5	Ch.1: 100; Ch.2: 10	200 none
	6	8	200 Frequency 80 Hz, pulse width 120 μ S for 1 sec. Return to program settings for 3 sec.
	7	Ch.1: 100; Ch.2: 8	200 none
	8	Ch.1: 100; Ch.2: 2	200 none
	9	Ch.1: 10; Ch.2: 2	200 none
Power Supply	9-volt battery		
Size	116 x 67 x 30mm		
Weight	180 grams with battery		

Distributed by